

Conform to EN 14683 Standard



REF	Mask Size	Type	Package
FMC	175x95mm	Non-sterile	50pcs/box, 50boxes/CTN 51x41x47cm

Anleitung

WORK 沃克®

MEDICAL FACE MASK

EN 14683 Type I

Instructions for use

- Confirm the nose clip in the top and the colored side in the front before putting on the face mask. Don't wear the face mask in reverse and touch the inside layer with unclean hands.
- Fully open the folded design and place a loop around each ear. Completely cover the mouth, nose and lower jaw, press the nose clip tightly, and stretch the ear loops appropriately to adjust the tightness, making the mask and the face fit properly.
- Hold both of the ear loops and remove the mask gently. Throw it into the medical waste container. The front of the mask is contaminated. Don't touch the front side of the mask during removing. Wash your hands immediately after removing the mask.

Warning

- Do not use if package has been previously opened or damaged.
- Avoid touching the inside of the mask with your hands.
- Single Use Only. The recommended use time is 4 hours.
- If the Face Mask is wet or contaminated by the patient's blood or body fluid, replace it in time.
- Dispose of used product in accordance with applicable regulations.
- Store in well-ventilated, dry place. Prevent heavy pressure, direct sunlight, hard objects.

REF : FMC
Size: 175*95MM

LOT
SEE THE BAG

Content: 25 pcs/bag

Hangzhou Shangyou Medical Equipment Co., Ltd.
ADD: No. 138, Louta Development Zone, Guancun Village, Louta Town, Xiaoshan District, Hangzhou, Zhejiang 311266, China

Lins Service & Consulting GmbH
ADD: Obere Sengasse 34/2, 69124 Heidelberg, Germany

Verpackung

